How to Program the VPOD Freedom







VirtuOx Inc. has received full accreditation by The Joint Commission as an Ambulatory Care Sleep Diagnostic Center



Getting Started

- After the patient has been scheduled in the VirtuOx portal, please navigate to "Print AOB/Upload" section on the left side bar
- Next to the patients name will be a red icon, indicating that this patient has been scheduled for a VPOD Freedom Overnight Oximetry, as shown in the image below
- For programming purposes, please have the "Reading ID" & "Patient ID" available for the patient that will be utilizing the VPOD Freedom. The reading ID can be found on the Virtuox pre-populated AOB and or Rx

Patie	nt List	t:																
Delet	Test Type	Readin ID	ng Patient Creation Rx Date Physicial Rx AOB Rx AOE Fax Uplo Name Date Name Name												Enter	Plac On Hold	r atie Instr	Rece Docs
×	New	103368	87 Feet, 04/15/2014 04/15/2014 Virtuox, 💥 💥 📄 📄							2				8		S		
×	New	103311	17 Freedom Test	,04/15/2014	04/15/2014	Virtuox, Dr.	×	×				3						S
×	New	1017										5						S
×	New	8624	This r	ed icon	indicate	s that	thi	s pa	atie	nt h	as	5				8		S
×	New	8137	been s	schedule	ed for a	wirele	55	test	. TI	his	red			Q		8		S
×	New	8000	for co	mmunic	ation fro	om the	VF	POD	Fre	edo	om,					8		6
×	New	8000	which	n is also	referred	l to as	a '	'han	dsh	nako	e".	5						S
×	New	8000							_			5				8		S
	2Î		1	6	I Page	e 1 0	f 1	8-> 1-1	20	~				tr o		Vie	ew 1 -	8 of 8

Beginning the Programming Process

 Turn the VPOD Freedom on by holding down the power button, located on the bottom left of the device, as shown in the image



Continuing The Programming Process





- Press button number 1 on the right side of the device shown in the image, this will open the menu on the screen
- Press button number 2 until the box is around the "G", which is the programming menu
- Press button number 1 to open the programming menu

Synchronizing the Reading ID



- Press button 2 until the second digit has the line under it, please note, the first digit should remain as a "0"
- In order to enter the reading ID, use button 1 to scroll through the numbers in the underlined column, when the desired number is reached, push button 2 to scroll to the next number
- After the reading ID has been entered properly, press button 2 for the line to be under the word " Check" then press button 1

Creating a Successful Handshake

- After pressing the word "Check", the Freedom will now say "Sending" on the screen
- This indicates that the wireless device is synchronizing with the reading ID that has been entered



Completing a Successful Handshake





- After the device has established a successful handshake through cellular signal, the patient ID will come back, as shown in the image
- Please wait for the blue "G" in the top right corner to disappear from the screen
- Press button 1 to select "ok"
- Press button 2 to scroll to the arrow, then select button 1, the Freedom will now be on the main screen

Ready for Patient Use

- After the handshake was successful, refresh the print AOB/Upload screen
- The icon that was previously red will now be orange, indicating that this device is now ready for patient use

Patie	nt List																	
Delet	Test Type	Reading ID	Patient Name Creation Rx Date Physicial Name Rx AOB Rx AOE Fax Uploi										Virtu Comi	Patie Com	Ente By	Place On Hold	Patie Instr	Rece Docs
×	New	1033687	Feet, Happy	Feet, 04/15/2014 04/15/2014 Virtuox, 🗶 🗶 🖃 📄											8		3	
×	New	1033117	Freedom, Test	reedom, 04/15/2014 04/15/2014 Virtuox, 🎉 🎉 📄 🔛 📓						5	R			8		S		
×	New	1017717	James, Lebron	03/26/2014	03/26/2014	Virtuox, Dr.	~	~				5				8		ø
×	New	862407														B		S
×	New	813700	The o	range ic	on now	shows	s th	at t	he	dev	ice	is 📘	12			8		6
×	New	800095	now s	sync'd w i	ith the p s now re	atient	t's i or u	info se.	rma	tior	n an	d	P			~		S
×	New	800087														8		S
×	New	800080	James, Lebron	05/20/2013	05/20/2013	Virtuox, Dr.	~	~	2		2	M				8		S
					ia Page	e 1 0	f 1	1-3- 3-1	20	~	NG 25					Vie	w 1 -	8 of 8

Transmitting the Data

Patie	nt List																	
Delet	Test Type	Reading ID	Patient Name	Creation Date	Rx Date	Physiciar Name	Rx	AOB	Rx PDF	AOE PDF	Fax PDF	Uplo: Appr	Virtu Comi	Patie	Enter By	Place On Hold	Patie Instr	Rece Docs
×	New	1033687	Feet, Happy	04/15/2014	04/15/2014	Virtuox, Dr.	×	×				5	12			~		B
×	New	1033117	Freedom, Test	,04/15/2014	04/15/2014	Virtuox, Dr.	×	×				5	X			8		100
×	New	1017717	James,	03/26/2014	03/26/2014	Virtuox,	1	4	2	N		1		2				ŝ
×	New	862407	The	areen	con sho	we the	at -			cef		5				3		S
×	New	813700	0	transm	ission h	as bee	en r	ece	ive	d		5				~		S
×	New	800095	Dr H									5				8		S
×	New	800087	Wade, Dwayne	05/20/2013	05/20/2013	Virtuox, Dr.	×	×	2		2	5	∎ ₽			8		1
×	New	800080	James, Lebron	05/20/2013	05/20/2013	Virtuox, Dr.	~	-		2		5				8		6
				50 S.	I Page	1 0	f 1		20	~	1921 - 12 1		in i	Si si		Vie	ew 1 -	8 of 8

- After the VPOD Freedom has been worn for 5 minutes or greater, the icon that was previously orange will now be green, this indicates a successful transmission of data
- This device will transmit data every hour on the hour

Approving a Wireless Test

- Select the green icon to open the approval window
- In this window the duration, various report types, and the start and end time can be viewed
- If the ONO session requirements have been met (*Data consisting of 30 minutes or greater*) and no further testing is required for this patient, please use the approve button to send the report to the referring physician
- If the ONO session requirements have not been met and further testing can benefit the patient, please use the close button to close the window
- Please contact the patient and advise them to the wear the device for another night, no further action will be required by the DME

Approve Wireless Test X								Approve Wireless Test											
 By selecting "Approve" the oximeter will no longer transmit this patient's oximetry data until the oximeter is reprogrammed. If you wish to test this patient longer, select "Close" and advise the patient to use the device again 								 By selecting "Approve" the oximeter will no longer transmit this patient's oximetry data until the oximeter is reprogrammed. If you wish to test this patient longer, select "Close" and advise the patient to use the device again. Reading ID: 1017717 Patient Name : James Lebron 											
use the d	evice aga	iin.					Summary	Detailed	Session	Start Time	End Time		D	uratio	n				
			Reading ID . 101771	7			Report	Report	Data										
			Patient Name : James L	ebron						03/26/2014 02:34:38 PM	03/26/2014 02:50:3	8 PM	C	:16:0	2				
Commence	Detailed	Contine	Chief Time	red Time	Duration					Approve	e								
Summary	Detailed	Session	Start Time	End Time	Duration			Trees			×	0.001	America	Contra	0.000				
Report	Report	Data				CS .		rype 1				I PUI	Appr	Com	Com				
			03/26/2014 02:34:38 PM	03/26/2014 02:50:38 PM	0:16:00	ults 1lts	*	New 103	Are you su	re you wish to approve this wire	less test:		5	R	9				
							×	New 101		Yes No		2	1						
			Approve	3			×	New 862					5	D					